



Australian Scientific Enterprise

Ph (+61) 2 9477 3870 | Fax (+61) 2 9477 7646
83 Hunter St Hornsby NSW 2077

ASE Cerebrospinal Fluid Control

Level 3

STORE REFRIGERATED (2-8°C)

CAUTION: *This material is prepared from human-based materials. Although source materials have been tested for certain diseases, no test can absolutely assure the absence of all infectious agents. Therefore, this product should be handled as a potential biohazard. For in vitro diagnostic use only.*

Intended Use

The ASE CSF controls are non-assay specific controls intended for professional use in monitoring the precision of the laboratory method for CSF Chemistry analytes including xanthochromia.

Preparation

Reconstitute the material with exactly 2mL deionised water.

Material Stability

The lyophilised product is stable until the expiry stated on the vial. Once reconstituted the material is stable for 7 days when stored tightly capped at 2-4°C in the dark.

Manufacturing Target Values

Batch # 21370	Target
Net Bilirubin Absorbance NBA (476nm)	0.010 ± 0.003**

***In the presence of haemoglobin*

Contact & Ordering Information

Please contact us with any queries or orders via the following channels:

Product Orders: Sales@ASEonline.com.au

Technical Support: Support@ASEonline.com.au

General Queries: Info@ASEonline.com.au

Or visit our website for our full product range:

www.ASEonline.com.au