



# Australian Scientific Enterprise

Ph (+61) 2 9477 3870 | Fax (+61) 2 9477 7646  
83 Hunter St Hornsby NSW 2077

## ASE Cerebrospinal Fluid Control NBA Negative

**STORE REFRIGERATED (2-8°C)**

**CAUTION:** *This material is prepared from human-based materials. Although source materials have been tested for certain diseases, no test can absolutely assure the absence of all infectious agents. Therefore, this product should be handled as a potential biohazard. For in vitro diagnostic use only.*

### Intended Use

The ASE CSF controls are non-assay specific controls intended for professional use in monitoring the precision of the laboratory method for CSF Chemistry analytes including xanthochromia.

### Preparation

Reconstitute the material with exactly 2mL deionised water.

### Material Stability

The lyophilised product is stable until the expiry stated on the vial. Once reconstituted the material is stable for 7 days when stored tightly capped at 2-4°C in the dark.

### Manufacturing Target Values

Batch # 21400	Target
Net Bilirubin Absorbance NBA (476nm)	0.004 ± 0.002

### Contact & Ordering Information

Please contact us with any queries or orders via the following channels:

Product Orders: [Sales@ASEonline.com.au](mailto:Sales@ASEonline.com.au)

Technical Support: [Support@ASEonline.com.au](mailto:Support@ASEonline.com.au)

General Queries: [Info@ASEonline.com.au](mailto:Info@ASEonline.com.au)

Or visit our website for our full product range:

[www.ASEonline.com.au](http://www.ASEonline.com.au)